










Canine Collection Procedure

Before collection, the application form must be fully completed. This form allows us to gather all the relevant owner, dog and collection officer details to process the payment, which must be made prior to any testing being completed.

Kit Contents



-  2 x Swabs
-  3 x Stick on barcodes (identical)
-  1 x Tamper evident bag
-  1 x Application form

Performing Collection

-  Try not to feed the dog for approximately 1 hour prior to collection.
-  Avoid any contact, especially licking, from other dogs prior to collection.
-  Always collect and package samples from one dog at a time.
-  Place 1 x barcode on application form and 1 x barcode on each of the swab packets.
-  See over for collection instructions.


Collection Checklist

Before sending your dog's sample, have you placed the following items in the reply paid envelope?

-  Swabs (placed back in separate packets, with barcode across the back and sealed inside the tamper proof evident bag).
-  Application form (fully completed, with barcode attached and payment details).

Collection Instructions

STEP 1

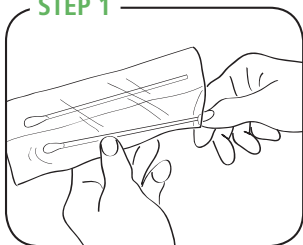

Carefully remove cotton swab from packet – avoid touching the cotton pad with your fingers.

STEP 2




Holding onto the shaft, insert the pad end of the swab into your dog's mouth.

STEP 3



Pinch the cheek around the swab and swirl around the pinched cheek for at least 10 seconds.

STEP 4

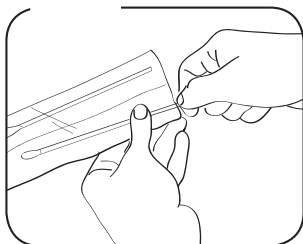

Carefully insert the swab (pad first) back into the packet and seal – avoid touching the cotton pad with your fingers. Repeat Steps 1-4 with second swab.

STEP 5

Place the barcode onto the sample packets and application form and place them all in the reply paid envelope provided and post.



The collector should take appropriate precautions to avoid injury.